




The Q105 and T105 ultrasonic cleaners can transfer the records from their internal memory on to a USB drive/USB stick...

For the cycle log to work correctly it is ESSENTIAL that the date and time are set correctly. If the clock is set incorrectly the machine has several safeguards built into the firmware which may flash up errors or not record cycles correctly.


Ensure that the USB logo and the Walker Electronics Logo are on the same side (see below picture) before pushing the adaptor and flash drive firmly together.

From the home screen press the  key twice. The following screen is displayed:



To return to the Home Screen press the  key. Press the  key and this screen is displayed:



To Cancel and return to the Home Screen press the  key

Unscrew the blue cap by turning it anti-clockwise. insert the USB flash drive and adaptor into the revealed socket.

When the USB stick is inserted, the log is copied to the stick as a text file.

After the transfer of data is complete the following screen is displayed:



Remove the USB flash drive and adaptor and replace the waterproof blue cover by turning it clockwise until light resistance is felt.

**NEVER LEAVE THE USB ADAPTOR AND DRIVE IN THE UNIT
AS THEY ARE NOT WATERPROOF**

You should now check that the logs have been correctly transferred by removing the adaptor and inserting the flash drive into a USB socket on a PC or laptop and inspecting the file.

The logs should then be transferred to a PC or laptop for permanent storage.


Once you are satisfied that they have been correctly transferred to the PC or laptop they may be deleted from the ultrasonic cleaner. We advise leaving the logs on the USB memory stick as a backup. Our standard memory stick can hold over one million individual cycle records.


Logs ARE NOT automatically deleted from the cleaner when transferred to the USB flash drive.


clearing the logs...

From the home screen press the  key three times. The following screen is displayed:



\$ is the total number of logs stored on the unit's internal memory. Press the  key to save the stored data. The display will then return to the home screen.

Press the  key to clear the stored data. The following screen is displayed:

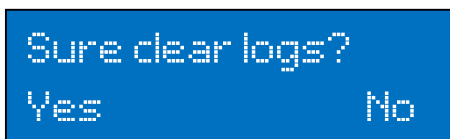

Press the  key to delete the stored data or the  key to save the data.

**ONCE THE LOGS ARE CLEARED FROM THE
ULTRASONIC CLEANER THEY ARE NOT RECOVERABLE!**

If you have any problems, contact a member of our team on 01636 892410.