

*The H105 ultrasonic cleaner can transfer the records from its internal memory to a USB drive/USB stick...*

**For the cycle log to work correctly it is ESSENTIAL that the date and time are set correctly. If the clock is set incorrectly the machine has several safeguards built into the firmware which may flash up errors or not record cycles correctly.**

If the USB and adapter are separated, align the two parts as in the picture below.



From the home screen press **Settings**

From the Main menu select Logs

From the Logs menu select Logs to USB drive and press **Select**



Unscrew the blue waterproof USB cap on the rear of the machine by turning it anticlockwise.  
Insert the USB stick and adaptor into the USB socket.

As soon as the stick is inserted the unit will save the data to the USB stick in the form of a Text file.

After the data has been copied remove the USB stick when prompted. Replace the blue waterproof USB cap on the rear of the machine by turning it clockwise.





You should now check that the logs have been correctly transferred by removing the adaptor and inserting the flash drive into a USB socket on a PC or laptop and inspecting the file.

The logs should then be transferred to a PC or laptop for permanent storage.

Once you are satisfied that they have been correctly transferred to the PC or laptop they may be deleted from the ultrasonic cleaner. We advise leaving the logs on the USB memory stick as a backup. Our standard memory stick can hold over one million individual cycle records.

Open the saved file and check that the data is correct  
BEFORE clearing the logs on the ultrasonic cleaner.  
The logs ARE NOT automatically deleted!


## *clearing the logs ...*

From the home screen press

From the Main menu select Logs

From the Logs menu select Delete logs

The following screen is then displayed

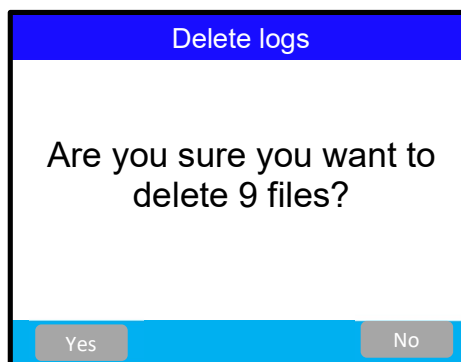

Press  to clear all the stored data or  to save the stored data and return to the Logs menu.

**WARNING. ONCE A LOG IS CLEARED IT IS NOT RECOVERABLE!**